


MEDICAL, RE-USEABLE FACE SHIELD

HEALTH CANADA CERTIFIED
MDEL 12444 (COMPANY ID 154615)



FSFA-WK
Medical Re-Useable
Face Shield

5,000+	\$3.93 /unit
100,000+	\$3.33 /unit

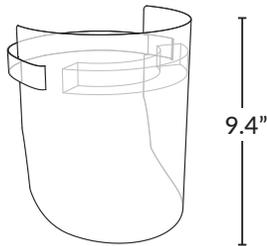


- ✓ 100% Made in Ontario, Canada
- ✓ Health Canada licensed
- ✓ Optically clear, anti-fog
- ✓ Full coverage sealed protection
- ✓ Skin-friendly foam
- ✓ One size fits all
- ✓ Re-useable & Recyclable
PET Plastic

Pull to fit any size, 1-touch release



ORDERS:
sales@hotpopfactory.com
(416) 837-8069

- ★ 100% Made in Canada
- ★ Health Canada Establishment License
- ★ Reusable & Recyclable
- ★ One Size Fits All

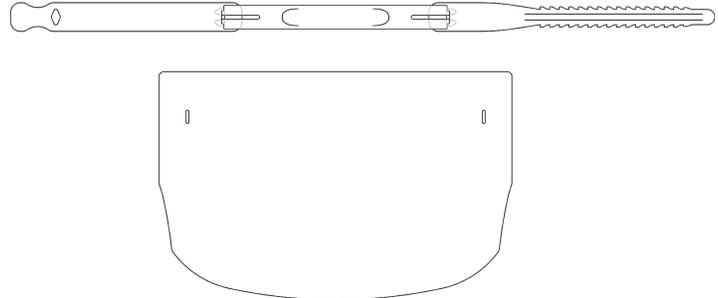

Product Description

The Worker's Fast Assembling Face Shield FSFA-WK is designed to protect against liquid splashes by covering the entire face including the eyes, nose and mouth area. The face shield is provided in flat-pack form and is easily assembled by the user.

Features

- Reusable x 10 times or more depending on user application
- Made from a lightweight, recyclable plastic material
- Optically clear, anti-fog plastic material that covers the entire human face.
- Durable headband that is easily adjustable to comfortably fit adult hat sizes from XS to XXL (21" - 26" head circumference).
- Can be worn for periods of up to 8-12 hours per day.
- Available Upon Request: Optional skin-friendly comfort pad made from a lightweight foam which prevents moisture absorption and has a soft flexible composition making it comfortable even for extended periods.

Technical Details

Materials:

Face Shield and Head Band: PET (Polyethylene Terephthalate)

Optional Skin-friendly Comfort Pad: 100% closed cell LDPF(Low Density Polyethylene Foam)

Dimensions:

Face Shield
Width: 370mm
Length: 239mm
Thickness: 0.5mm

Head Band (Assembled)
Width: 30mm
Length: 518mm - 658mm
Thickness: 0.75mm

Optional Comfort Pad
Width: 134mm
Length: 25mm
Thickness: 3.0mm

Sterilization

Not sterilized, clean before use. See instructions included.

Packaging

100 units per corrugated board carton of face shields, includes assembly, proper care and use instructions in bilingual (French/English).

Contact Us



215 Spadina Ave. Suite B05, Toronto, ON, M5T2C7



416 306 6468



info@hotpopfactory.com

Cleaning your face shield

- ⚠ Face shields delivered to facilities should be considered **non-sterile**. Clean and sanitize this product before every use as per institutional guidelines.

The Fast Assembling Face Shield is constructed of durable PET plastic, which can be cleaned with most hard surface disinfectants listed by Health Canada:

<https://www.canada.ca/en/health-canada/services/drugs-health-products/disinfectants/covid-19/list.html> We recommend regular cleaning with warm water and mild soap, and disinfecting with a 70% isopropyl alcohol solution.

- ⚠ CAUTION: If any part becomes broken, discoloured, or scratched, discard the device.
- ⚠ CAUTION: If the shield is exposed to anyone with flu-like symptoms or otherwise contaminated, remove immediately and replace with a new unit.

Cleaning with soap & water

1. Disassemble the Fast Assembling Face Shield (refer to *Disassembly and Reassembly* on page 5).
2. With a soft, clean, lint-free cloth, wipe all surfaces of the product with a solution of warm water and mild soap, no hotter than 60 degrees Celsius.
3. Allow to air dry completely, or fully dry with a clean, absorbent towel.

Refer to *Disassembly and Reassembly* on page 6 for assembly instructions before your next use.

Disinfecting with isopropyl alcohol (70%)

1. Clean and sanitize the work area.
2. Wash hands with soap and water for at least 20 seconds, then don clean gloves.
3. Use a clean cloth saturated with mild soap and warm water to carefully wipe the inside, followed by the outside of the face shield.
4. Use a clean cloth saturated with 70% isopropyl alcohol to carefully wipe all surfaces and features of the headband, followed by the inside of the face shield, and finally its outside surface.
5. Allow to air dry completely, or fully dry with a clean, absorbent towel.

If any cleaning product residue is left on the shield after cleaning, carefully wipe with 70% isopropyl alcohol to remove.

Refer to *Disassembly and Reassembly* on page 5 for assembly instructions before your next use.

Warnings

- ⚠ The shield is not impact-resistant. Use in combination with appropriate eye protection where needed.
- ⚠ Do not use near sparks or open flame. Failure to follow this warning may result in serious harm or disfigurement.

Contact Us



215 Spadina Ave. Suite B05, Toronto, ON, M5T2C7



416 837 8069



sales@hotpopfactory.com